Wisbi®

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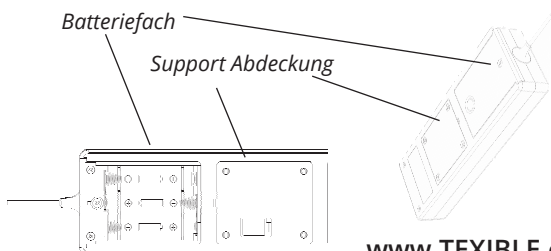
www.TEXTIBLE.com

SAFETY INFORMATION

Please carefully read through these instructions before using the device! TEXIBLE GmbH assumes no liability for material damages and personal injuries that arise due to non-observance of the operating instructions and especially the safety information!

TEXIBLE Wisbi® is not a medical device and must not be used to diagnose or treat illnesses. Ask your doctor for prevention, diagnosis and treatment of illnesses.

The TEXIBLE Wisbi transmitter must be placed under or next to the bed in a way that there is no danger for the patient through the cable.

Note for care workers

TEXIBLE Wisbi is a support aid for care workers and does not serve as a replacement. TEXIBLE Wisbi must be checked for functionality before each use.

Maintenance and troubleshooting

TEXIBLE Wisbi is free of maintenance except when the battery needs replacement. Only tighten the screws on the battery compartment slightly.

The following steps should be considered if the function is restricted:

- Is the bed pad correctly connected and the transmitter switched on?
- Is the TEXIBLE Wisbi transmitter with external contact correctly connected to the call system with a 3.5mm jack plug?
- Is the transmitter battery functioning?
- Is the bed pad free of defects?

If the mentioned points are fulfilled, but the function is still limited, please contact the technical staff.

Note: The TEXIBLE Wisbi transmitter with external contact is equipped with a stereo Relais, which switches an NO and an NC relais in the event of an alarm. The transmitter may only be connected to the call system via a 3.5mm jack plug stereo, it is important to pay attention to the polarity of the plug.

USER MANUAL

1. PACKAGE CONTENTS

The package contents:

- 1x TEXIBLE Wisbi® – Bed Pad
- 1x TEXIBLE Wisbi® Plus transmitter with external contact

2. SCOPE OF APPLICATION AND FUNCTION

TEXIBLE Wisbi is the innovative nursing care product that stands out for its ability to automatically signal an alarm when moisture, body fluids or a bed exit are detected.

The sensor textile is used in a wide variety of nursing applications, such as senior and nursing homes, assisted living and family care. The necessary basic care, as well as the incontinence care and the supervision of demented and fall-endangered residents represents an increased care effort for all persons involved.

TEXIBLE Wisbi offers the right support here. The system allows an optimal skin climate and minimizes the risk of running away.

3. PUTTING TEXIBLE WISBI INTO OPERATION

The TEXIBLE Wisbi transmitter PLUS with external contact must be connected to the call system via a 3.5mm jack plug stereo.

1. **INSERT BATTERIES:** Insert three mignon batteries (AA) into the battery compartment. When the batteries are inserted correctly, the white LED flashes briefly.

Switch the TEXIBLE Wisbi transmitter on through pressing the button  once. The white LED flashes.

2. **CONNECTION BED PAD:** Place the bed pad as the top layer on the sheet. White cotton side up, blue side down. Then fasten the TEXIBLE Wisbi transmitter using the push buttons with the bed pad. (See image below)

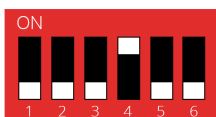
3. CONNECT TRANSMITTER WITH A CALL SYSTEM:

The TEXIBLE Wisbi transmitter PLUS with external contact must be connected to the call system via a 3.5mm jack plug stereo. The device is equipped with a stereo relais. A measurement is taken every 2 seconds. When detecting a wet bed insert or leaving the bed, the stereo relais is switched NO and NC for approx. 500ms and an alarm is sent.

4. ALARM SETTINGS OF TEXIBLE WISBI TRANSMITTER:

The alarm settings (occupancy detection ON/OFF, delays) of the transmitter can be customized to your Needs:


To do so, open the 4 screws on the back Support coverage. The address can then be changed in the inspection window via the DIP switch (Number 5 and 6).


Alarm WET and OCCUPIED immediately

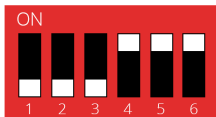Alarm WET immediately and OCCUPIED deactivated

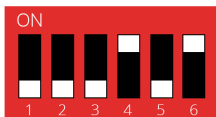Alarm WET immediately and OCCUPIED 5 minutes delayed

Alarm WET immediately and OCCUPIED 15 minutes delayed

Then close the support cover again.

Functional test

TEXIBLE Wisbi must be checked for proper functioning before each use. Switch on the TEXIBLE Wisbi transmitter. Press the button  for 3 seconds until the orange LED flashes. This will lead to a functional test of the electronics and sends an alarm.

Switching TEXIBLE Wisbi off

Once TEXIBLE Wisbi is switched on it is constantly in standby mode and measures every 2 seconds.

If the transmitter is not used, it can be switched off by pressing the button  for 3 seconds to preserve the batteries.

By pressing the button  again once, the transmitter is switched on again.

Changing batteries of TEXIBLE Wisbi

Open the rear cover of the battery compartment by loosening the screw. Insert three mignon batteries (AA) into the battery compartment as shown and close the cover again.

4. OPERATING CONCEPT

Function	Button	LED Display
Switching TEXIBLE Wisbi Transmitter ON	 Press 1 x	flashing white
Switching TEXIBLE Wisbi Transmitter OFF	 Press 3 sec	glowing red
Functional test	 Press 3 sec	flashing orange
Alarm		flashing orange
Battery low		red & orange flash alternately
Error		glowing red
Sending data		flashing orange

5. DISPOSAL INFORMATION

Dispose the TEXIBLE Wisbi transmitter with accompanying batteries at a designated collection point.

The TEXIBLE Wisbi bed pad can be disposed in household waste. Please contact your waste disposal company or the representative for more information about nearby collection points.

Dispose of the packaging material in the collection containers for cardboard, paper and plastics.

6. WARRANTY

We rectify any device defects due to material or manufacturing faults by repair or replacement free of charge within the statutory warranty period. The warranty expires in the case of intervention by third parties.

7. EC - DECLARATION OF CONFORMITY

Manufacturer:

TEXIBLE GmbH, Widagasse 9, 6850 Dornbirn, Austria

The following guidelines have been complied with:

EMC 2014/30/EU, RED 2014/53 EU, LVD 2014/35/EU, RoHS 2011/65/EU

The declaration of conformity can be requested from the manufacturer.

8. CUSTOMER SERVICE


Please contact the manufacturer or your dealer if malfunctions still occur despite proper handling or if the device has been damaged.

9. TECHNICAL DATA TEXIBLE WISBI

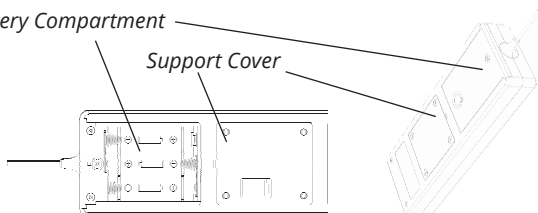Size	64 x 100cm
Weight	250g
Water absorption	up to 700ml/m ²
Trigger quantity	10ml
Minimal body weight (for occupancy detection)	45kg
Material	67% Cotton 30% Polyester 3% Polyurethane
Wash resistance	
Power supply	3xAA mignon batteries, LR6 =====
Power	1,4mW
Protection class	IP40
Ambient temperature	0 to +55°C
Battery life	6 month depending on application
Connection of external alarm contact	3,5mm jack plug stereo
Triggering of external alarm contact	NO and NC relais
Connexion jack plug stereo	NO: Tip-Sleeve NC: Ring-Sleeve

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TEXIBLE GmbH, Widagasse 9, 6850 Dornbirn, Oostenrijk